

PATIENT INFORMATION BROCHURE

LAPAROSCOPIC HERNIA MESH FIXATION DEVICE

- 1. DESCRIPTIVE
INFORMATION**
 - 2. POTENTIAL
RISKS & BENEFITS
INFORMATION**
 - 3. ADDITIONAL
INFORMATION**
-



LIQUIFIX FIX8™

LAPARO HERNIA MESH FIXATION DEVICE

LIQUIFIX FIX8™

LAPARO HERNIA MESH FIXATION DEVICE


1. DESCRIPTIVE INFORMATION

What is the purpose of LIQUIFIX FIX8™ Laparoscopic?

LIQUIFIX FIX8™ Laparoscopic Hernia Mesh Fixation device is intended for use in laparoscopic (keyhole) surgical repair of groin (inguinal and femoral) hernias, achieved through the fixation of prosthetic polypropylene or polyester mesh to the abdominal wall and the approximation of the peritoneum.

A hernia is when an internal part of the body pushes through a weakness in the abdominal wall. To repair a hernia through surgery, the hernia contents or other protruding tissues are returned to their normal position and the weakness in the abdominal wall is repaired. Sometimes it is necessary to support the repair with a mesh which is fixed to the abdominal wall at the site of the hernia. The mesh is secured until it becomes integrated with the surrounding tissue. This is often done with the use of sutures (stitches) or small screw-like-tacks, which are designed to penetrate the tissue and hold the mesh in place. LIQUIFIX FIX8™, which is a medical grade tissue adhesive (glue), is another option.

About LIQUIFIX FIX8™ Laparoscopic

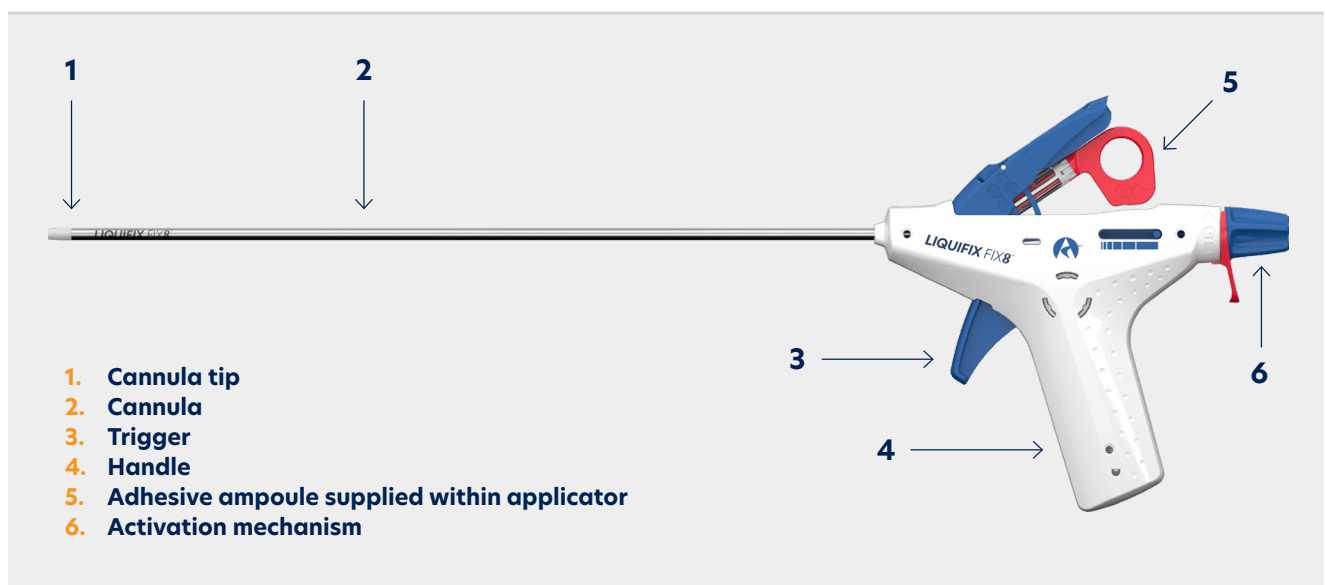

The LIQUIFIX FIX8™ Laparoscopic Hernia Mesh Fixation device consists of adhesive (n-butyl-2-cyanoacrylate) supplied in a glass vial within a handheld, single-use, disposable applicator. After the surgeon has primed the device, the device releases a drop of adhesive (anchor) when the trigger is pulled and released. Each adhesive drop is 12.5mg on average. The adhesive is applied to an implanted hernia repair mesh in order to fix the mesh to the abdominal wall. When the adhesive is applied to the mesh and tissue, it sets (forms a chemical bond) due to the moisture on the tissue surface within approximately 10 seconds and allows the mesh to remain in the correct position. LIQUIFIX FIX8™ can also be used to close the peritoneum (a membrane that lines the abdominal wall), which is cut to access the abdomen when the TAPP procedure is used.

When should LIQUIFIX FIX8™ not be used (contraindications)?

LIQUIFIX FIX8™ should not be used in individuals with a hypersensitivity (allergy) to cyanoacrylate adhesives, formaldehyde, or D&C Violet No. 2 dye.

Your doctor will assess if your hernia defect is suitable for the hernia repair procedure with LIQUIFIX FIX8™ and guide you on all relevant information related to the device.

2. POTENTIAL RISKS & BENEFITS INFORMATION

What are the potential benefits to the patient for the use of LIQUIFIX FIX8™ Laparoscopic?

LIQUIFIX FIX8™ provides atraumatic/non-penetrative fixation. This may reduce the risk of mechanical tissue trauma and also allows for fixation over sensitive anatomical locations, when compared to traditional penetrative fixation devices such as tackers or sutures.

A pivotal study conducted with LIQUIFIX FIX8™ in the US has shown the following benefits:

- ▶ **Non-inferior mesh fixation and peritoneal closure to a tacker device**
- ▶ **Non-inferior hernia recurrence rate at 6 months post-hernia repair when compared to a tacker device**
- ▶ **Less incidence of complications such as hematoma (collection of blood outside blood vessels) and seroma (a collection of fluid at the surgical site) when compared to a tacker device**
- ▶ **Allows for mesh fixation over areas considered too sensitive for mechanical fixation such as nerves and blood vessels**

What are the potential risks to the patient for the use of LIQUIFIX FIX8™ Laparoscopic?

The potential adverse effects (e.g. complications) associated with the use of the LIQUIFIX FIX8™ device may include, but are not limited to, toxic reaction and allergic reaction.

As with the majority of implanted devices, adverse reactions associated with the use of the device may include transient local irritation at the implant site (i.e. inflammation or discomfort at the site of surgery, caused by a reaction to an irritant substance, lasting for a short time only) and a transitory inflammatory foreign body response (i.e. a non-permanent reaction caused by an object that is not typically found in the body).

The US pivotal clinical study has been performed with LIQUIFIX FIX8™ Laparoscopic Hernia Mesh Fixation device. There were no increased safety risks with the use of LIQUIFIX FIX8™ than when a tacker device was used. The following possibly device-related adverse events were observed with LIQUIFIX FIX8™ in a US clinical study of 284 patients. All these adverse events were either possibly or definitely related to the laparoscopic hernia repair procedure, and therefore not necessarily a direct result from the device.

- ▶ **Hernia recurrence: 0.7% of subjects had this effect**
- ▶ **Seroma: 13.4% of subjects had this effect**
- ▶ **Groin pain: 2.8% of subjects had this effect**
- ▶ **Neuralgia: 1.4% of subjects had this effect**
- ▶ **Hypoesthesia: 0.7% of subjects had this effect**
- ▶ **Hematoma: 0.7% of subjects had this effect**
- ▶ **Intestinal obstruction: 0.7% of subjects had this effect**
- ▶ **Mesh infection: 0.7% of subjects had this effect**
- ▶ **Lymphadenitis: 0.7% of subjects had this effect**
- ▶ **Testicular pain: 1.4% of subjects had this effect**
- ▶ **Swelling: 0.7% of subjects had this effect**
- ▶ **Genital Hemorrhage: 0.7% of subjects had this effect**
- ▶ **Spermatic cord inflammation: 0.7% of subjects had this effect**
- ▶ **Orchitis: 0.7% of subjects had this effect**
- ▶ **Muscle Strain: 0.7% of subjects had this effect**

Your doctor can discuss with you the potential adverse events associated with laparoscopic hernia repair surgery, as well as any alternatives available to you.

What should I expect throughout the surgical procedure?

Your doctor can discuss the procedure with you. A brief summary of what to expect is below:

Before the procedure, your doctor will confirm the presence of hernia and that you are appropriate for hernia repair surgery. The surgical procedure will be minimally invasive and performed as a standard of care procedure under sterile conditions in an operating room. The surgery is normally performed under general anaesthesia. Your abdomen will be inflated with harmless gas to give the doctor a better look at your organs. Small incisions (cuts) are made in the lower part of the abdomen, and a camera (laparoscope) is inserted into the abdomen to visualise the hernia defect. The hernia sac is removed from the defect in the abdominal wall, a hernia mesh is placed over the hernia defect and the mesh is fixed using LIQUIFIX FIX8™.

In TAPP surgery, access to the hernia defect is created by incising the peritoneum and then placing a mesh through that incision to cover the hernia defect. In this case, LIQUIFIX FIX8™ adhesive may be used to fix the mesh and also close the peritoneum incision.

At the end of the surgery, your abdomen will be deflated and the small abdominal incisions are then closed with standard wound closure devices such as sutures. You should discuss all hernia repair options with your doctor to determine which approach is best for you.

Your doctor will let you know what to expect after your hernia repair surgery. The majority of patients undergoing elective or non-emergent groin hernia repair go home the same day as the surgery. Many patients return to normal activity soon after the procedure, however, your doctor can help you determine when you can resume physical activity once evaluated at your post-operative visit.

You do not need to perform any device maintenance after surgery in relation to the LIQUIFIX FIX8™ adhesive.

General warnings and precautions

You should talk to your doctor about all potential risks and adverse events discussed in this brochure, as well as any other concerns you have.

Non-clinical testing has demonstrated that LIQUIFIX FIX8™ adhesive implant is Magnetic Resonance Imaging (MRI) safe; it can safely be used in a magnetic environment.

Expected failure time and mode and its effect on the patient

The in-use lifetime of the implanted adhesive is approximately 2 weeks. The adhesive is only required to temporarily hold the mesh in place until the mesh is integrated with the abdominal wall. The adhesive does not need to be removed after this time and, as it does not degrade, it will remain within the body.

Should the LIQUIFIX FIX8™ fail, i.e. does not maintain bond before tissue integration, then you may experience symptoms of hernia recurrence (e.g. groin lump, persistent groin pain) and you should contact your doctor for examination to diagnose the appropriate medical condition.

Any follow-up after the procedure should be as recommended by the operating clinician.

3. ADDITIONAL INFORMATION

Clinical Studies Experience

A 284 patient US pivotal study was performed to support the clinical safety and effectiveness of LIQUIFIX FIX8™ in the US. The study compared difference in pain from baseline (worst pain experienced within 1 month of screening) to 6 months post-hernia repair with a US-marketed tacker device. The results showed that the improvement in pain in LIQUIFIX FIX8™ was not inferior to AbsorbaTack™ (control device).

The rate of hernia recurrence at 6 months for LIQUIFIX FIX8™ was not inferior to the control device.

Successful hernia mesh fixation was accomplished in all hernias for LIQUIFIX FIX8™ and control device. The rate of successful hernia mesh fixation for LIQUIFIX FIX8™ was not inferior to the control device. The rate of successful peritoneal closure at time of surgery for LIQUIFIX FIX8™ was not inferior to the control device.

The Quality of Life (Carolinas Comfort Scale Total Score) had a decreasing trend (improvement to Quality of Life) from 1 week through 12 months follow-up in both LIQUIFIX FIX8™ and control.

The incidence of device-related adverse events by subject were comparable in the LIQUIFIX FIX8™ and control groups.

There were no unanticipated adverse events.

A summary of additional Outside-US (OUS) studies that are supportive of the clinical safety and effectiveness of LIQUIFIX FIX8™ have briefly been summarized below:

Hernia repair (TAPP/TEP) surgery using LIQUIFIX FIX8™ adhesive technique is suitable for mesh fixation and closure of the peritoneal incision.

- ▶ **In a 20 patient TAPP inguinal hernia repair prospective clinical study, no recurrence was reported up to 3-month post-operative follow-up¹.**
- ▶ **In a 34 patient TAPP inguinal and femoral hernia repair prospective clinical study, no recurrence was reported up to 12-month post-operative follow-up².**
- ▶ **In a 67 patient TAPP/TEP inguinal hernia repair prospective clinical study, one recurrence (1.5%) was reported at 1-day post-surgery. No other recurrence was reported up to 12-month post-operative follow-up³.**
- ▶ **In a 196 patient TAPP inguinal and femoral hernia repair retrospective clinical study, one recurrence (0.5%) was reported up to 24-month post-operative follow-up⁴.**
- ▶ **In a 152 patient TAPP/TEP inguinal hernia repair retrospective study using an iron-loaded mesh, there were no confirmed cases of recurrence at minimum 6-month follow-up⁵.**
- ▶ **In a 10 patient TAPP inguinal hernia repair prospective study, there were no recurrent hernias, wound infections or hematomas reported up to 30-day post-operative follow-up⁶.**
- ▶ **In a 25 patient TAPP inguinal hernia repair prospective study where LIQUIFIX FIX8™ was used to close the peritoneum, there were no intraoperative complications reported⁷.**

IRM 06 0340 04a

Date of issuance: 2023-08-04



GLOSSARY

Carolinas Comfort Scale (CCS) - A specific questionnaire used to evaluate Quality of Life in patients who underwent abdominal hernia repair with mesh.

Hemorrhage - Loss of blood from damaged blood vessels.

Hernia - The bulging of an internal organ through a weak area or tear in the muscle or other tissue that holds it in place.

Hematoma - A pool of mostly clotted blood that forms in an organ, tissue, or body space.

Hypoaesthesia - Partial or total loss of sensation in a part of your body.

Intestinal Obstruction - A partial or complete block of the small or large intestine that keeps food, liquid, gas, and stool from moving through the intestines in a normal way.

Lymphadenitis - Enlargement in one or more lymph nodes, usually due to infection.

MRI - Magnetic Resonance Imaging is a non-invasive procedure in which radio waves and a powerful magnet linked to a computer are used to create detailed pictures of areas inside the body.

Neuralgia - Pain in a nerve pathway.

Orchitis - Inflammation of the testicles.

Seroma - A mass or lump caused by a build-up of clear fluid in a tissue, organ, or body cavity.

TAPP - Laparoscopic Inguinal Hernia Repair (TAPP) is keyhole surgery to repair a groin hernia. Transabdominal pre-peritoneal (TAPP) surgery is performed through the peritoneum lining of your abdominal cavity.

TEP - Laparoscopic Inguinal Hernia Repair (TEP) is keyhole surgery to repair a groin hernia. Totally extra peritoneal (TEP) surgery repairs your hernia without entering the perineal cavity.



Advanced Medical Solutions

Advanced Medical Solutions Ltd
Western Wood Way,
Langage Science Park,
Plymouth, Devon,
PL7 5BG, United Kingdom

T: +44 (0) 1752 209 955

E: info@liquiband.com

- Mittermair R, Jenic G, Kolenik R, Sorre C. TAPP surgery with mesh fixation and peritoneal closure using n-butyl-2-cyanoacrylate (LiquiBand FIX8®)—initial experience. *Eur Surg*, 2016; 48, 110.
- Dauser B, Szyszkowitz A, Seitinger G, Herbst F. Fixation of mesh and peritoneal closure using n-butyl cyanoacrylate following laparoscopic inguinal hernia repair. *Eur Surg* (2016). doi:10.1007/s10353-016-0450-0
- Schmidt, J. Fixation of Titanized mesh with N-Butyl-Cyanoacrylate (Nbca) Using a Novel Device: Biocompatibility and Short Term Results in Laparoscopic Hernia Repair. *JSM Gen Surg Cases Images* (2017) 2(2):1026.
- P. Wilson, L. Hickey. Laparoscopic transabdominal preperitoneal (TAPP) groin hernia repair using n-butyl-2-cyanoacrylate (LiquiBand FIX8®) for mesh fixation and peritoneal closure: learning experience during introduction into clinical practice.
- Özveri E, Şanlı DET, Yıldırım D, Gök H, Ertem M. Magnetic resonance visualization of iron loaded meshes in patients with pain after inguinal hernia repair [published online ahead of print, 2020 Mar 12]. *Hernia*. 2020;10.1007/s10029-020-02168-9. doi:10.1007/s10029-020-02168-9
- Koprivica, R. et Al. Adhesive Techniques for Mesh and Peritoneum Fixation in Laparoscopic Inguinal Hernia Repair. *Surgery and Surgical Endoscopy*, Vol 2, No 2, Oct 2020, pages 11-16
- Bhoopat T, Chansaenroj P. Comparison of intraocular pressure during laparoscopic totally extraperitoneal (TEP) versus transabdominal preperitoneal (TAPP) inguinal hernia repair. *Surg Endosc*. 2022 Mar;36(3):2018-2024. doi: 10.1007/s00464-021-08487-x. Epub 2021 Apr 12. PMID: 33844088.